Data Sheet



THERMARMOUR Baby Wrap

Product Code	OB202-55-57TAEM55	
GTIN/UDI	05060885279682	
Colour(s)	Emergency Blue Outer with White Inner	
Size	55cm x 57cm (can be cut to size if required)	
Material	Non-conductive bilaminate foil & needle punched polyester fleece	
Care	Patient Specific, cannot be laundered	
Buy Online	www.interweavetextiles.com/products/patient-warming/	

Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

Key Features	 Latex Free, Recyclable & X-ray Compatible. Easily transportable standalone device. No waiting to use once opened; a bonus for first responders. A cost and time efficient passive warming method.
--------------	--

Usage

Open packaging and remove the THERMARMOUR Baby Wrap. Unfold the wrap fully, placing the infant in the middle as shown. Wrap and secure with the foil side outwards.







Packing

Manufactured in our UK facility, THERMAMOUR is a readily available, sustainable solution.

Per Carton	40, Individually Bagged
Carton Size	40 x 32 x 30cm
Pallet Quantity	2160

Address: Interweave House, Old Power Way, Lowfields Business Park, Elland, HX5 9DE Tel: 01422 411463 | sales@thermarmour.com | www.thermarmour.com



THERMARMOUR is a trading brand of Interweave Textiles Ltd. Interweave Textiles Ltd is a company registered in England & Wales: 05267141


Data Sheet



Storing and Handling

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact sales@thermarmour.com before use.

Technical Drawing

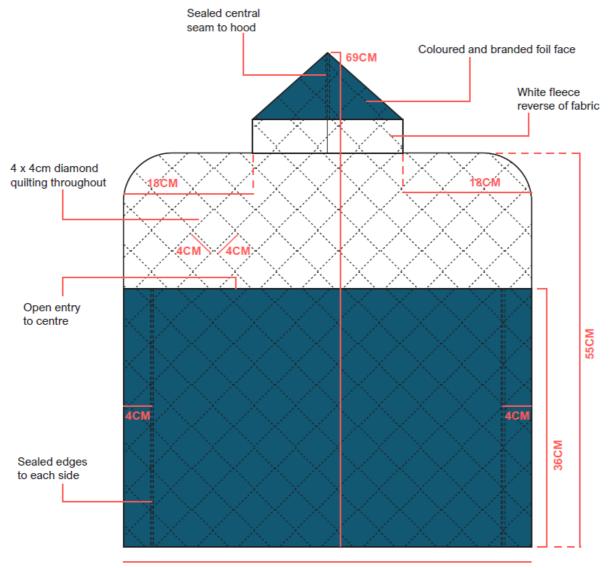

57CM

Risks and Precautions

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

If there are any safety concerns with this device, please inform THERMARMOUR and MHRA.

Address: Interweave House, Old Power Way, Lowfields Business Park, Elland, HX5 9DE Tel: 01422 411463 | sales@thermarmour.com | www.thermarmour.com



THERMARMOUR is a trading brand of Interweave Textiles Ltd. Interweave Textiles Ltd is a company registered in England & Wales: 05267141