

## Data Sheet

**THERMARMOUR®**

### THERMARMOUR Military Poncho

<b>Product Code</b>	OB210LGE-----OVEM55
<b>GTIN/UDI</b>	05060885279279
<b>Colour(s)</b>	Olive Green Outer with White Inner
<b>Size</b>	One size fits most (can be cut to size if required)
<b>Material</b>	Non-conductive bilaminate foil & needle punched polyester fleece
<b>Care</b>	Patient Specific, cannot be laundered
<b>Buy Online</b>	<a href="http://www.interweavetextiles.com/products/patient-warming/">www.interweavetextiles.com/products/patient-warming/</a>

Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

#### Key Features

- ❖ Latex Free, Recyclable & X-ray Compatible.
- ❖ Easily transportable standalone device.
- ❖ No waiting to use once opened; a bonus for first responders.
- ❖ A cost and time efficient passive warming method.

#### Usage

Open packaging and remove the THERMARMOUR Poncho. Unfold the poncho fully, and if required, cut to the desired size. Place poncho over the distressed party's head with the foil side outwards.

#### Storing and Handling

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact [sales@thermarmour.com](mailto:sales@thermarmour.com) before use.

#### Risks and Precautions

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

If there are any safety concerns with this device, please inform THERMARMOUR and MHRA.



Address: Interweave House, Old Power Way, Lowfields Business Park, Elland, HX5 9DE  
Tel: 01422 411463 | [sales@thermarmour.com](mailto:sales@thermarmour.com) | [www.thermarmour.com](http://www.thermarmour.com)

THERMARMOUR is a trading brand of Interweave Textiles Ltd. Interweave Textiles Ltd is a company registered in England & Wales: 05267141

**interweave**  
healthcare